


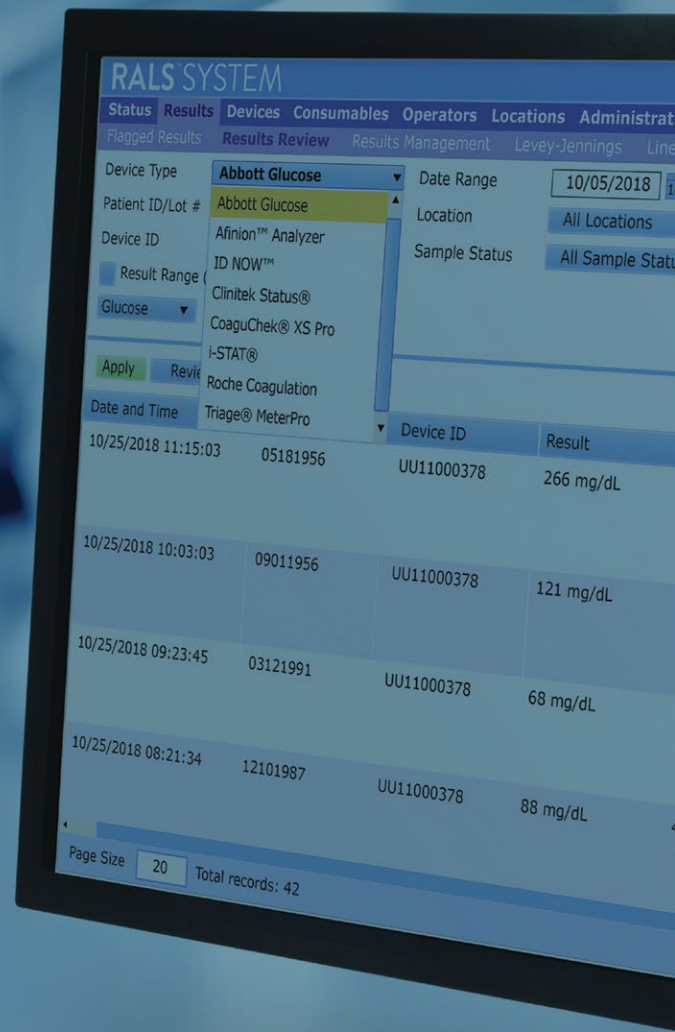

DEVICE CONNECTIVITY FOR
POINT-OF-CARE TESTING

ABBOTT GLUCOSE CONNECTIVITY

Abbott FreeStyle Precision Pro® and Precision Xceed Pro® glucose meters are the first and only hospital-grade blood glucose and β -ketone monitoring systems among leading hospital brands to deliver multiple key patient safety features in one comprehensive solution.



Connectivity by the RALS System enables users to:


- Eliminate the need for device-specific data managers
- Centralize all POC test results into the RALS System
- Transfer results to an EMR/EHR/LIS
- Email results to operators and managers
- Access and manage mis-identified patient and out-of-range QC results



VISIT RALS.COM

RALS™ | Abbott Glucose Connectivity

HOSPITAL / FACILITY / INSTITUTION NETWORK

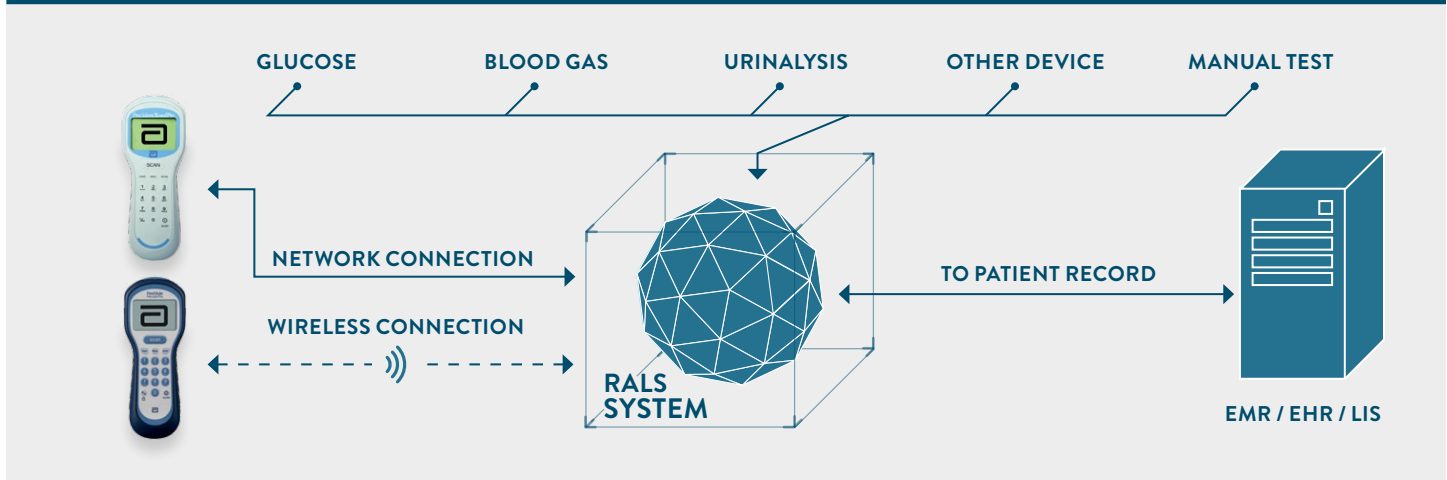

DATA FLOW

Abbott glucose meters transfer data to RALS via RALS Remote Connect software, directly to a network, through a Dataport, or wirelessly (Abbott FreeStyle Precision Pro only) where it is evaluated and either sent directly to the EMR/EHR/LIS or held for review.

DIRECT-TO-DEVICE INTERFACING

Abbott FreeStyle Precision Pro and Precision Xceed Pro glucose meters transfer a wide range of data wired or wirelessly (Abbott FreeStyle Precision Pro only) to RALS, including:

- Patient test results
- QC test results
- Operator ID
- Date/Time
- Location
- Comments
- Linearity
- Proficiency

HOW THE DEVICE CONNECTS

- RALS Remote Connect (RRC)
- Direct network
- Wirelessly (Abbott Freestyle only)
- Dataport

BI-DIRECTIONAL COMMUNICATION

From device to RALS system:

- Results (Patient, QC, Linearity and Proficiency)
- Patient ID and Operator ID

From RALS system to device:

- Patient ID and Operator ID
- Device Configuration

AVAILABLE REPORTS FROM THE RALS SYSTEM

- Anomalous Results by Device
- Anomalous Results by Operator
- Institution Statistics
- Location Statistics
- LIS Statistics by Location
- LIS Statistics by Operator
- Operator ID Compliance by Location
- Operator Pending Recertification Date
- Operator Pending Recertification Range
- Operator Statistics
- Patient ID Compliance by Operator
- Patient ID Compliance by Location
- Patient Results by Operator
- Patient Results by Patient
- QC Test Results
- Device Statistics Summary
- Levey-Jennings
- Corrective Action Compliance by Location and by Operator
- Patient Result Comments
- Comment Codes
- Test Materials
- Workload Summary
- Operators with No Certifications
- RALS Device Log

TO LEARN MORE, VISIT RALS.COM

© 2019 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. INFO246597 AI-COLL-0061 Rev 2 06/19.

