



DEVICE CONNECTIVITY FOR
POINT-OF-CARE TESTING

RALS™ DEVICE INTERFACING

The Stat Profile Prime® analyzer combines the revolutionary microelectronics of the consumer world with Nova Biomedical's innovative MicroSensor Card™ technology for a smaller, faster, more powerful yet simpler and less expensive analyzer.

Connectivity by the RALS System enables users to:


- Centralize all test results into the RALS System
- Transfer results to an EMR/EHR/LIS
- Email results to operators and managers
- Access and manage mis-identified patient and out of range QC results
- Monitor and manage operator certifications



VISIT RALS.COM

RALS™ | Stat Profile Prime® Analyzer Connectivity

HOSPITAL / FACILITY / INSTITUTION NETWORK

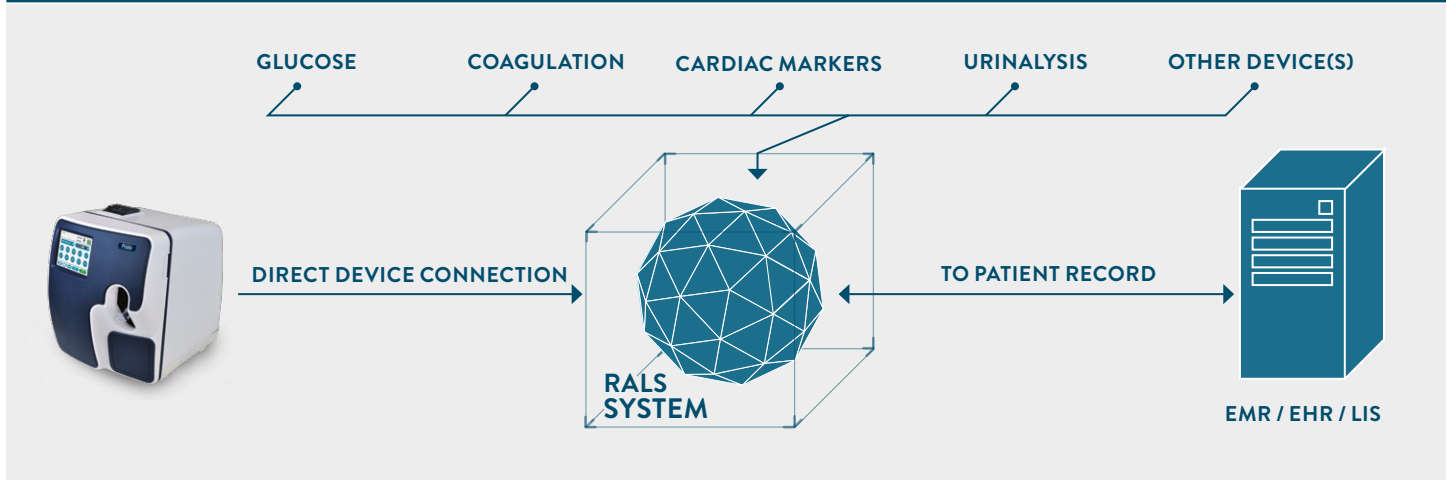

DATA FLOW

The Stat Profile Prime® Analyzer transfers data to the RALS System, where it is evaluated and either sent directly to an EMR/EHR/LIS or held for review.

DIRECT-TO-DEVICE INTERFACING

The Stat Profile Prime® analyzer transfers a wide range of data to the RALS System, including:

- Patient test results
- QC test results
- Proficiency test results
- Operator ID
- Date/Time
- Location
- Comments
- Specimen Type (Patient)

UNI-DIRECTIONAL COMMUNICATION

From device to RALS system:

- Patient test results
- QC test results
- Proficiency test results
- Operator ID
- Date/Time
- Location
- Comments
- Specimen Type (Patient)

AVAILABLE REPORTS FROM THE RALS SYSTEM

- Anomalous Results by Device
- Anomalous Results by Operator
- Institution Statistics
- Location Statistics
- LIS Statistics by Location
- LIS Statistics by Operator
- Operator ID Compliance by Location
- Operator Pending Recertification Date
- Operator Pending Recertification Range
- Operator Statistics
- Patient ID Compliance by Operator
- Patient ID Compliance by Location
- Patient Results by Operator
- Patient Results by Patient
- QC Test Results
- Device Statistics Summary
- Levey-Jennings
- Corrective Action Compliance by Location and by Operator
- Patient Result Comments
- Comment Codes
- Test Materials
- Workload Summary
- Operators with No Certifications
- Patient Result Comments with Clean Meter Status
- RALS Devices Log

HOW THE DEVICE CONNECTS

- Direct Network

TO LEARN MORE, VISIT RALS.COM

© 2019 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. INFO216824 AI-COLL-0046 Rev 2 05/19

