



DEVICE CONNECTIVITY FOR  
POINT-OF-CARE TESTING

# RALS™ DEVICE INTERFACING

The HEMOCHRON® Response is a whole blood coagulation monitoring instrument designed to be used at the point-of-care. The Response includes an easy-to-read screen and data management features, including patient and operator ID options. QC and operator lockout can be enabled for enhanced regulatory compliance.

## CONNECTIVITY TO RALS ENABLES USERS TO:


- Centralize all test results into the RALS system
- Transfer results to an EMR/EHR/LIS
- Email results to operators and managers
- Access and manage mis-identified patient and out of range QC results



VISIT [RALS.COM](http://RALS.COM)

# RALS™ | Hemochron® Response Interfacing

## HOSPITAL / FACILITY / INSTITUTION NETWORK

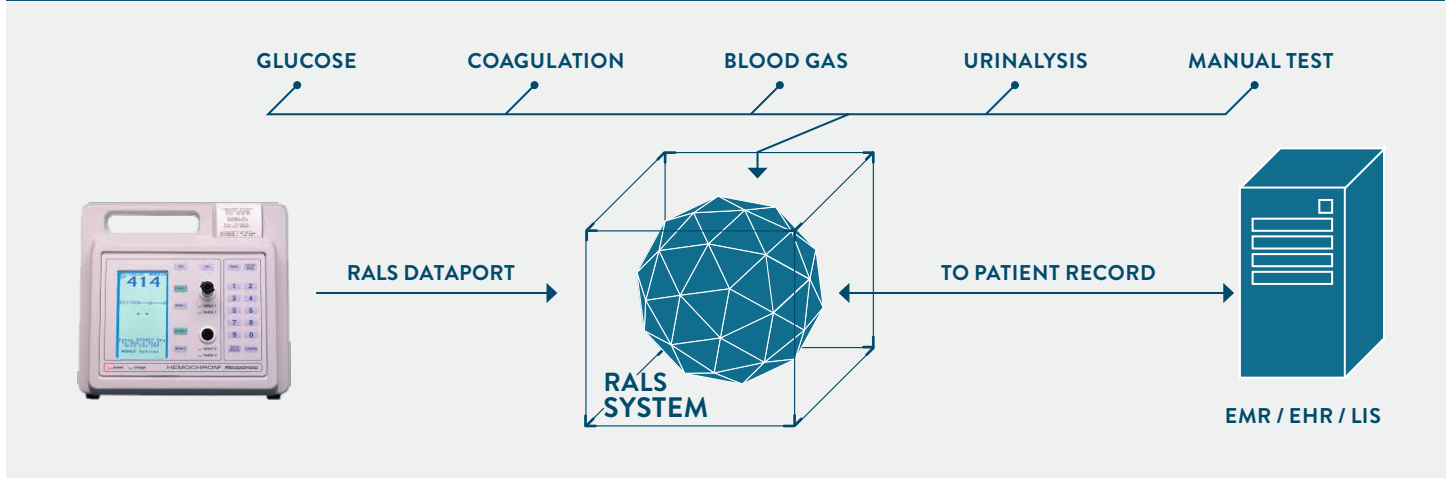

## DATA FLOW

The Hemochron® Response transfers data to RALS via a RALS dataport where it is evaluated and either sent directly to the EMR/EHR/LIS or held for review.

## UNI-DIRECTIONAL COMMUNICATION

Data sent from the device to RALS

- Patient test results
- QC test results
- Calibration test results
- Operator ID
- Date/Time
- Comments
- Materials

Data sent from RALS to the device

- Operator Lists
- Comment Lists

## AVAILABLE REPORTS FROM THE RALS SYSTEM

- RALS™ Devices Log
- Workload Summary
- Anomalous Results by Device
- Anomalous Results by Operator
- Operator Pending Recertification Range
- Operator Pending Recertification Date
- Operators with No Certifications
- Operator Statistics
- Patient Results by Operator
- Patient Results by Patient
- LIS Statistics by Location
- LIS Statistics by Operator
- Institution Statistics
- Location Statistics
- Levey-Jennings
- QC Test Results
- Device Statistics Summary
- Test Materials

TO LEARN MORE, VISIT [RALS.COM](http://RALS.COM)

© 2019 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. AI-COLL-0120 Rev. 1 12/19

