



DEVICE CONNECTIVITY FOR
POINT-OF-CARE TESTING

RALS™ DEVICE INTERFACING

The HemoPoint® H2 measures hemoglobin levels and calculates hematocrit level results using capillary, venous or arterial blood samples. It delivers accurate results for hemoglobin and an estimated hematocrit level using only a single drop of blood by utilizing advanced microcuvette technology.

CONNECTIVITY TO RALS ENABLES USERS TO:


- Centralize all test results into the RALS system
- Transfer results to an EMR/EHR/LIS
- Email results to operators and managers
- Access and manage mis-identified patient and out of range QC results



VISIT [RALs.COM](https://www.rals.com)

RALS™ | HemoPoint® H2 Meter Interfacing

HOSPITAL / FACILITY / INSTITUTION NETWORK

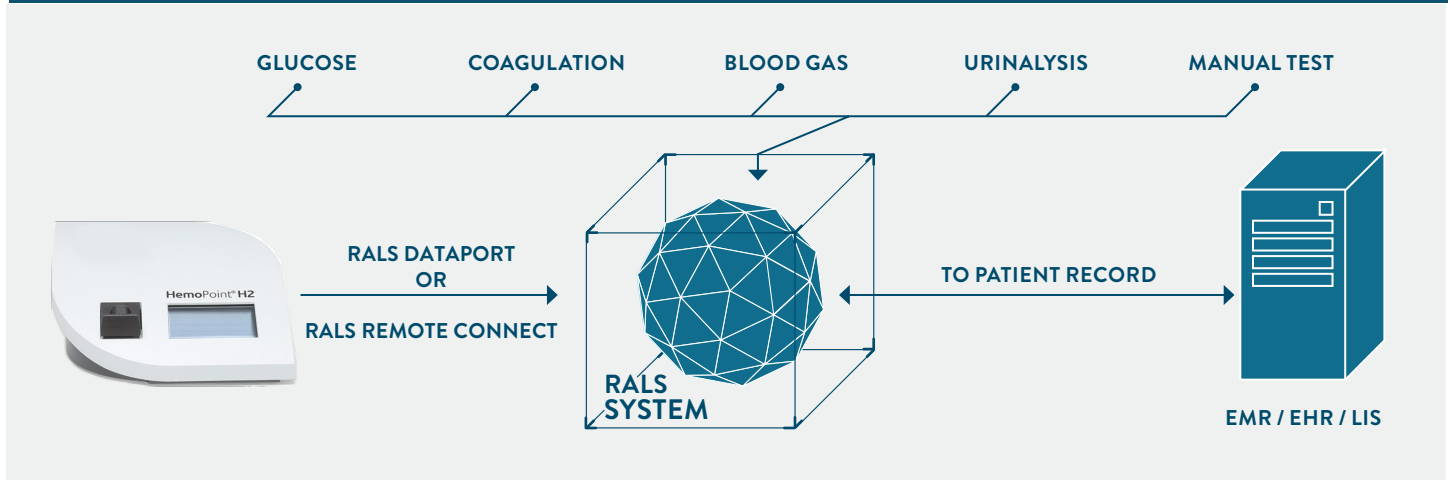

DATA FLOW

The HemoPoint H2 connects to RALS via a RALS Dataport or RALS Remote Connect (RRC) software, where data is evaluated and either sent directly to the EMR/EHR/LIS or held for review.

UNI-DIRECTIONAL COMMUNICATION

Data sent from the device to RALS

- Date and time
- HGB value with units
- HCT value with units (if enabled on device)
- Critical limits if a patient result identified on device
- Firmware version
- Serial number

AVAILABLE REPORTS FROM THE RALS SYSTEM

- RALS™ Devices Log
- Workload Summary
- Anomalous Results by Device
- Anomalous Results by Operator
- Operator Pending Recertification Range
- Operator Pending Recertification Date
- Operators with No Certifications
- Operator Statistics
- Patient Results by Operator
- Patient Results by Patient
- LIS Statistics by Location
- LIS Statistics by Operator
- Institution Statistics
- Location Statistics
- Levey-Jennings
- QC Test Results
- Device Statistics Summary
- Test Materials

TO LEARN MORE, VISIT [RALS.COM](https://www.rals.com)

© 2019 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. Any person depicted in such photos is a model. AI-COLL-0118 Rev. 1 12/19

